SE

Funktion

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Supports and stabilises the wrist.

Indication

Over-exertion and/or inflammation of the hand or forearm.

Materials

SBR Neoprene.

Application

- Place the support on the wrist.
- Close the Velcro band.
- Make sure the support sits firmly without pressing.

Important Information

- If you have questions regarding application, please contact your local dealer or the prescriber.
- The product is not flameproof, avoid temperature above 120°C/ 248°F.
- Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- Re: neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- If the product contains latex or rubber and you are allergic to latex/ rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- If you are allergic to heat, we advise you to use a product that not contains neoprene.

*The prescriber must be a qualified doctor, PT, OT or Orthotist.

Washing Instructions

- Close all Velcro.
- Remove detachable splints.
- Mesh laundry bag is recommended.
- Stretch product while wet and let dry.

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